

# ***THE ULTIMATE 1000 LIFT™***

## **Instructions for Installing Freedom Pivot Machine to Trolley OP-015 Key RP-017G/SD Key**

**MORTUARY LIFT™ COMPANY**

***A L i f t i n g E x p e r i e n c e ™***




127 Cottage Grove Avenue S.E.  
Cedar Rapids, Iowa 52403  
800-628-8809 • 319-364-4910  
[www.mortuarylift.com](http://www.mortuarylift.com)


# *THE ULTIMATE 1000 LIFT™*

## Instructions for Installing Freedom Pivot Machine to Trolley and Track System

1. Slide the trolley into the track.

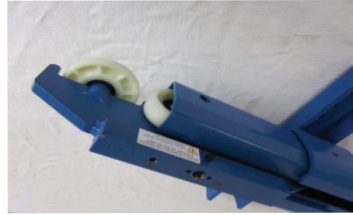

2. Assemble the trolley bumpers to the 9/16" holes at each end of the track and tighten the 1/2" nut.

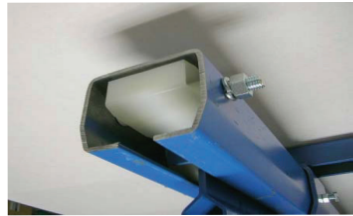

**Failure to assemble the trolley stop bumpers may result in the machine falling from the track causing severe injury or even death.**

3. Assemble and tighten the green strut channel to the track using the large U-bolts provided with the machine. The strut channel sections should be set at 36" centers.




**Note:** Be sure that the trolley is placed between the two sections of the strut channel.

4. Place the machine under track section, plug the Ultimate 1000 lift into an outlet and run the straps out of the machine by depressing the “Down” button. When the straps are long enough to reach the green strut channel, **STOP**.


5. Carefully flip the machine over and then fasten all four corner strap hooks onto the end holes of the green strut channel. Press the “Up” button and begin to raise the machine toward the track.



6. Place thread lock on the 5/8-13x4” bolt and center the machine so that you can slip the spindle key into the trolley key way.

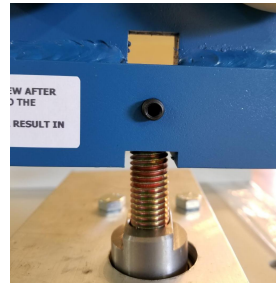

7. When positioning the pivot allen stop bolts make sure they are on the correct side of the safety posts. All Three must be on the same side as the black electrical conduit box.

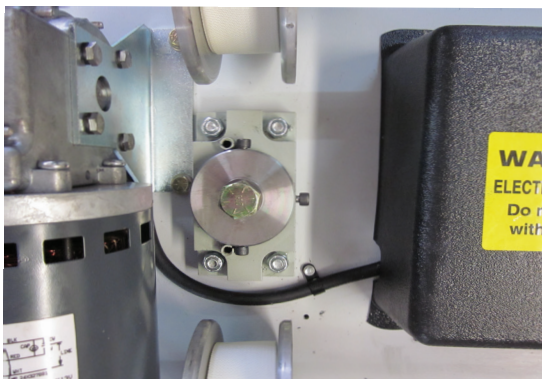

8. Once you have the machine centered on the trolley screw in the 5/8 - 13x4" pivot bolt and tighten down.




9. Make sure to tighten set screw on trolley to the pivot bolt (Do not over tighten).



### 10. Pivot Rotation Instructions

#### **NEVER REMOVE ALL THREE SET SCREWS**

- To rotate the pivot 90 degrees, remove one screw.
- To rotate the pivot 180 degrees, remove two screws.
- The screws you remove will determine which directions the pivot will rotate

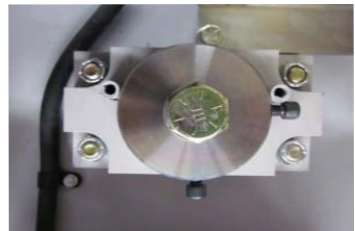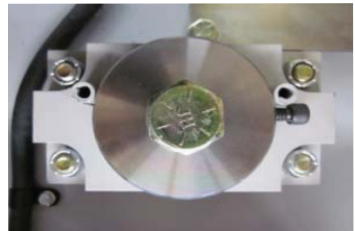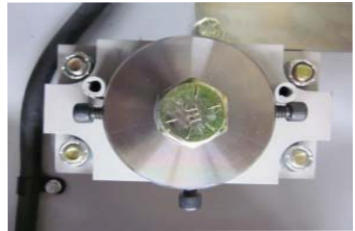

#### **WARNING:**

**NEVER REMOVE ALL THREE SET SCREWS FROM THE PIVOT DEVICE. THIS CAN CAUSE THE MACHINE TO FALL FROM THE TRACK, CAUSING POSSIBLE INJURY OR EVEN DEATH!**

MORTUARY LIFT™ COMPANY

— BUILDING EXPERIENCE —



**THE ULTIMATE 1000 LIFT™**

127 Collage Grove Avenue S.E.  
Cedar Rapids, Iowa 52403  
800-628-8809 • 319-364-4910  
www.mortuarylift.com

Rev 7.30.2018